UTM®

Universal medium for collection, transport, and preservation of





Copan Universal Transport Medium (UTM®) system is intended for the collection, transport and preservation of clinical specimens containing Viruses, Chlamydia, Mycoplasma and Ureaplasma.



FLOQSwabs®




Ensure a quick, capillarity-driven sample uptake and a superior elution of the biological specimen, expanding downstream diagnostic testing capabilities.



Different fill volumes and screw cap *formats*



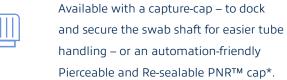

UTM® is available in 1 or 3mL fill volumes.

Glass beads



Three glass beads in each tube facilitate the release and dispersion of patient material and virus particles from the swab during vortexing.



Skirted conical tube

12x80mm or 16x100mm freestanding shatterproof polypropylene conical tubes ensure efficient centrifugation.

Preservation

UTM® Performance

Copan UTM® is **CE-IVD** and **510(k)** cleared, and is **compliant** with **CLSI M40-A2** standards. UTM® **preserves the viability** of all the target organisms tested **for 48 hours** at both room and refrigerated temperature.

UTM® unique formulation includes antibiotics and antimycotics to prevent overgrowth of bacterial and fungal flora without affecting viruses, chlamydia, mycoplasma, or ureaplasma viability.

According to the vast scientific literature and on the information provided by diagnostic platform manufacturers, UTM® can be used for downstream detection of:

- o Nucleic acids^{1,2,3}
- Antigens^{4,5}
- Rickettsiae⁶

FLOQSwabs®

Cut out for everyone

FLOQSwabs® offer variable sizes, diameters, breaking points and tip shapes to be used in plenty of applications. This made FLOQSwabs® a well-tolerated alternative to invasive, painful, and costly collection procedures^{7,8}

Do you have a specific application in mind? Choose the right FLOQSwabs®!



Fields of application

Preanalytics made different



Respiratory Diseases

Regular, minitip and flexible minitip



STI & HPV

Regular and minitip



Cutaneous Infections

Regular



Veterinary

Regular and minitip

Laboratoru

Easy handling

Samples collected with Copan UTM-RT® can be stored at room or refrigerated temperature for 48 hours, or frozen if processing is delayed. The capture cap and the skirted tube ease the handling of the specimen, while the tubes fits most laboratory equipment. The glass beads provide a fast elution of the sample from the swab and the conical tube ensures an efficient centrifugation.




Laboratory

Versatile processing

UTM® can be processed both manually and with automated laboratory platforms as Copan UniverseTM. Copan UTM® is widely present in scientific literature and, based on information provided by the manufacturers of the diagnostic test, it can be used with:

- Antigen detection assays 7,8,9
- Molecular assays^{10,11,12}
- High-throughput systems^{3,13,14}

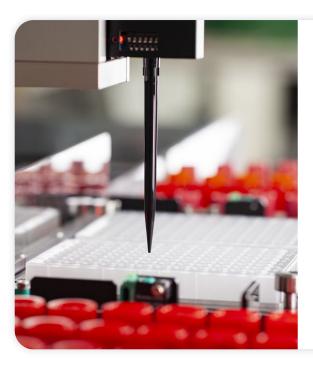

UniVerse™


Flexible and open solution for molecular testing sample preparation


With UniVerse™, you can automate sample preparation for molecular testing: tube decapping and recapping, vortexing, spinning, barcode identification, and liquid transfer to secondary tubes or 96-well plates. UniVerse™ handles indiscriminately different tube sizes and different shaft types without requiring to remove the swab from the tube. With its four different operational modes, UniVerse™ integrates impeccably into your lab's workflow through a 2-way LIS exchange.

Pooling samples, saving time and resources

PodSwab™ - UTM® for pooling

PodSwab™ is a 6 ml UTM® tube paired with five FLOQSwabs® that allows for pooling up to five different individual samples in one single tube.

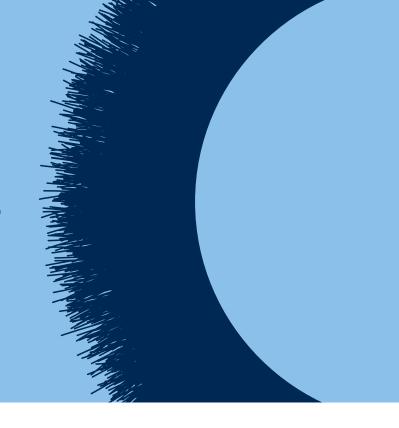

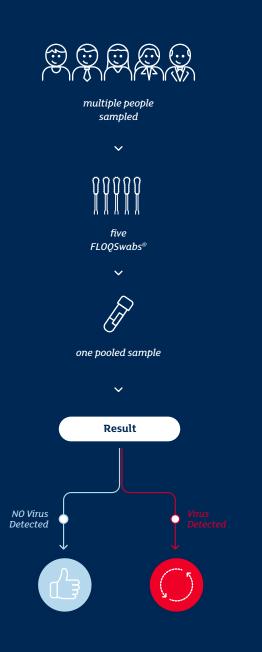

How pooling works

The future of COVID-19 screening

If the pooled sample tests negative, all the pooled patients are considered negative – by performing one test instead of 5. Conversely, if the pooled sample is positive, each person within the pod must be individually retested.

PodSwab™ offers many operational advantages, especially in low-incidence diseases: testers can significantly increase the volume of people tested while reducing the cost per single test, in the respect of full sample and patient traceability.



Ordering information

Choose between different tube sizes and medium fill volumes, in bulk packs or in combination with either FLOQSwabs® or polyester fiber swabs.

Kits of UTM® with CLASSIQSwabs™

Cat N.	Description	Pack size	Sample*
302C	16x100mm tube filled with 3ml UTM® medium + 2 CLASSIQSwab™ with polyester applicator and molded breaking point	300 pieces (6 boxes of 50 kits)	Combined body site sampling

Kits of UTM $\mbox{with FLOQSwabs}\mbox{\ensuremath{\mathbb{R}}}$

Cat N.	Description	Pack size	Sample*	
305C	16x100mm tube filled with 3ml UTM® medium + 1 flexible minitip FLOQSwabs® with molded breaking point	300 pieces (6 boxes of 50 kits)	Eye, ear, nasal passages, nasopharynx, throat, urogenital tracts and pediatric sites	
307C	16x100mm tube filled with 3ml UTM® medium + 1 minitip FLOQSwabs® with molded breaking point	300 pieces (6 boxes of 50 kits)	Eye, ear, nasal passages, nasopharynx, throat, urogenital tracts and pediatric sites	
321C	16x100 mm tube filled with 3 ml UTM® medium + 1 regular FLOQSwabs® with molded breaking point + 1 flexible minitip FLOQSwabs® with molded breaking point	300 pieces (6 boxes of 50 kits)	combined body site sampling	
346C	16x100mm tube filled with 3ml UTM® medium + 1 regular FLOQSwabs® with molded breaking point	300 pieces (6 boxes of 50 kits)	Nose, throat, vagina, rectum, faeces and wounds	

Cat N.	Description	Pack size	Sample*	
355C	12x8omm tube filled with 3ml UTM® medium + 1 regular FLOQSwabs® with molded breaking point	300 pieces (6 boxes of 50 kits)	Nose, throat, vagina, rectum, faeces and wounds	
358C	12x8omm tube filled with 3ml UTM® medium + 1 flexible minitip FLOQSwabs® with molded breaking point 300 pieces (6 boxes of 50 kits)		Eye, ear, nasal passages, nasopharynx, throat, urogenital tracts and pediatric sites	
359C	12x8omm tube filled with 1ml UTM® medium + 1 regular FLOQSwabs® with molded breaking point 300 pieces (6 boxes of 50 kits)		Nose, throat, vagina, rectum, faeces and wounds	
360C	12x8omm tube filled with 1ml UTM® medium + 1 flexible minitip FLOQSwabs® with molded breaking point 300 pieces (6 boxes of 50 kits)		Eye, ear, nasal passages, nasopharynx, throat, urogenital tracts and pediatric sites	
361C	12x8omm tube filled with 1ml UTM® medium + 1 minitip FLOQSwabs® with molded breaking point	300 pieces (6 boxes of 50 kits)	Eye, ear, nasal passages, nasopharynx, throat, urogenital tracts and pediatric sites	
365C	12x8omm tube filled with 2ml UTM® medium + 1 flexible minitip FLOQSwabs® with molded breaking point 300 pieces (6 boxes of 50 k		Eye, ear, nasal passages, nasopharynx, throat, urogenital tracts and pediatric sites.	
366C	12x8omm tube filled with 2ml UTM® medium + 1 regular FLOQSwabs® with molded breaking point 300 pieces (6 boxes of 50 kits)		Nose, throat, vagina, rectum, faeces and wounds	
368C	12x8o mm tube filled with 1 ml UTM® medium + 1 regular FLOQSwabs® with molded breaking point + 1 flexible minitip FLOQSwabs® with molded breaking point	300 pieces (6 boxes of 50 kits)	combined body site sampling	

Cat N.	Description	Pack size	Sample*
	16x100 mm tube filled with 6ml UTM® medium + 5 regular FLOQSwabs® with molded breaking point		
3E076N05		300 pieces (6 boxes of 50 kits)	Nose, throat, buccal
	WELL TANK OF THE PROPERTY OF T		

UTM® tubes in bulk

Cat N.	Description	Pack size	
330C	16x100mm tube filled with 3ml UTM® medium WIRA 6T wages defined by the control of the control o	300 pieces (6 boxes of 50 pieces)	
331C	25x90 mm tube filled with 10ml UTM® medium UTM-RT transport medium for: Vivese, Chlamydia, Bycoplasma 4 Utropiama News.	90 pieces (6 boxes of 15 pieces)	
348C	16x100mm tube filled with 2,5ml UTM® medium	300 pieces (6 boxes of 50 pieces)	
349C	16x100mm tube filled with 1.5ml UTM® medium	300 pieces (6 boxes of 50 pieces)	
350C	12x8omm tube filled with 1ml UTM® medium	300 pieces (6 boxes of 50 pieces)	
353C	12x8omm tube filled with 3ml UTM® medium	300 pieces (6 boxes of 50 pieces)	

Scientific references

All the independent studies we cited in this product focus are listed here.

- 1. Rogers AA et al (2020) Evaluation of Transport Media and Specimen Transport Conditions for the Detection of SARS-CoV-2 by Use of Real-Time Reverse Transcription-PCR. J Clin Microbiol 58:e00708-20.
- 2. Tian J et al (2021) Identification of Viruses in Patients With Postviral Olfactory Dysfunction by Multiplex Reverse-Transcription Polymerase Chain Reaction. Laryngoscope 131:158-164.
- 3. Folgueira L et al (2019) Comparison of the Panther Fusion and Allplexassays for the detection of respiratory viruses in clinical samples. PLoS One 14(12): e0226403
- 4. Bruzzone B et al (2021) Comparative diagnostic performance of different rapid antigen detection tests for COVID-19 in the real-world hospital setting.

 Int J Infect Dis S1201-9712(21)00384-2
- 5. Baro B et al (2021) Performance characteristics of five antigen-detecting rapid diagnostic test (Ag-RDT) for SARS-CoV-2 asymptomatic infection: a head-to-head benchmark comparison. J Infect So163-4453(21)00191-2.
- 6. Frickmann H et al (2013) Comparison of different media for preservation and transport of viable rickettsiae. Eur JMicrobiol Immunol 3:194-197
- 7. David J. Speicher, Kathy Luinstra, Emma J. Smith, et al. Non-invasive detection of IgG antibodies from common pathogenic viruses using oral flocked swabs. Diagnostic Microbiology and Infectious Disease, 2020.
- 8. Carolynn DeByle, Lisa Bulkow, Karen Miernyk, et al. Comparison of nasopharyngeal flocked swabs and nasopharyngeal wash collection methods for respiratory virus detection in hospitalized children using real-time polymerase chain reaction. Journal of Virological Methods, 2012.
- 9. Bianco G et al (2021) Evaluation of an antigen-based test for hospital point-of-care diagnosis of SARS-CoV-2 infection. J Clin Virol 10438
- 10. Corman V et al (2021) Comparison of seven commercial SARS-CoV-2 rapid point-of-care antigen tests: a single-centre laboratory evaluation study.

 Lancet Microbe
- 11. Benirschke RC et al (2019) Clinical Impact of Rapid Point-of-Care PCR Influenza Testing in an Urgent Care Setting: a Single-Center Study. J Clin Microbiol 53:e01281-18
- 12. Shakya S et al (2018) Prevalence of Sexually Transmitted Infections among Married Women in Rural Nepal. Infect Dis Obstet Gynecol. 2018:4980396
- 13. Peretz A et al (2020) Mycoplasma and Ureaplasma carriage in pregnant women: the prevalence of transmission from mother to newborn. BMC Pregnancy and Childbirth 20:456
- 14. Procop GW et al (2020) A Comparison of Five SARS-CoV-2 Molecular Assays With Clinical Correlations. Am J Clin Pathol 155(1):69-78
- 15. Gomes da Costa V et al (2019) Molecular and serological surveys of canine distemper virus: A meta-analysis of crosssectional studies. PLoS ONE 14(5): e0217594
- 16. Danilenko DM et al (2021) Antigenic and Genetic Characterization of Swine Influenza Viruses Identified in the European Region of Russia, 2014–2020. Front. Microbiol. 12:662028.
- 17. Jevsnik M et al (2020) Detection of herpes simplex and varicella-zoster virus from skin lesions:comparison of RT-PCR and isothermal amplification for rapid identification. Diagn Microbiol Infect Dis 97(2):115015.
- 18. Poljak M et al (2020) Clinical Evaluation of the cobas SARS-CoV-2 Test and a Diagnostic Platform Switch during 48 Hours in the Midst of the COVID-19 Pandemic. J Clin Microbiol 58: e00599-20
- 19. Bordi L et al (2020) Rapid and sensitive detection of SARS-CoV-2 RNA using the Simplexa™ COVID-19 direct assay. J Clin Virol 128: 104416.
- 20. Park WB, et al. Virus Isolation from the First Patient with SARS-CoV-2 in Korea. J Korean Med Sci, 2020 Feb 24;35(7):e84

^{*} The PNR cap is still under development



This document may contain product information otherwise not accessible or valid in your country. Please be aware that Copan Italia S.p.A. does take any responsibility for accessing such information which may not comply with any valid legal process, regulation, registration or usage in the country of your origin. Product clearance and availability restrictions may apply in some Countries. Please refer to Copan website (www.copangroup.com) to view and/or download the most recent version of the brochure. This document is mainly int ended for marketing purposes, always consult product insert for complete information. The use of this product in association with diagnostic kits or instrumentation should be internally validated by the user. ©2021 Copan Italia. All rights reserved. The trademarks mentioned herein are property of Copan Italia S.p.A.

Code: JMKPF003R01

